



This guide is for iOS 10+ iPhone users.



AbiBird®

How to


Pause AbiSensor Notifications

You can pause an AbiSensor's notifications – perhaps your cared-for person is away for the weekend.

To pause an AbiSensor's notifications, you need to:



1. Access AbiSensor's graph page
2. Pause notifications

Step 1: Access AbiSensor's graph page

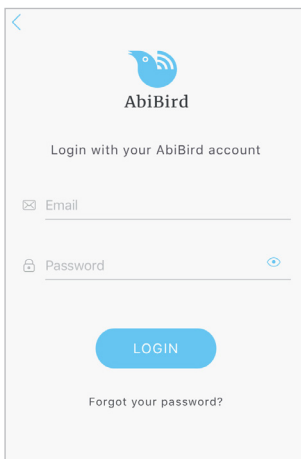

Login to your AbiBird app with your username (email) and password. The *Devices* page will open.

NOTE: If you are already in the AbiBird app you can access the *Devices* page by paging back (arrow < top left) until the *Devices* page opens.

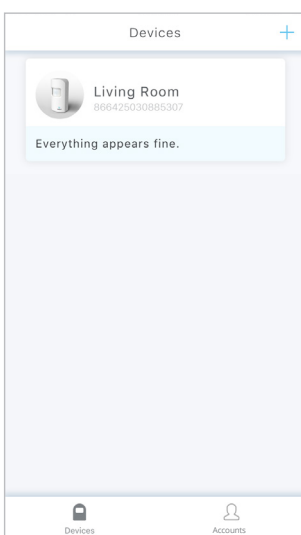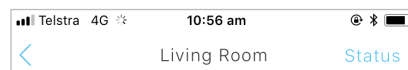

From the *Devices* page, select the AbiSensor for which you want to pause notifications.


The AbiSensor's graph page will open.



This guide is for iOS 10+ iPhone users.



Step 2: Pause notifications

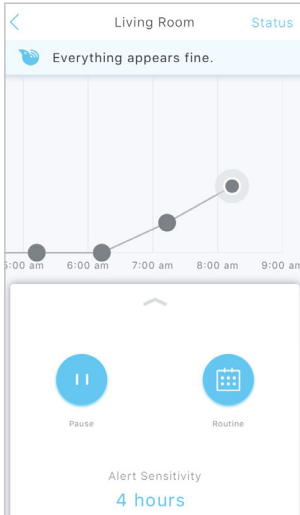


The AbiSensor's graph page has the name of the AbiSensor at the top of the page.

From the panel at the bottom of the AbiSensor's graph page, select **Pause**. The **Pause** icon changes state to **Resume**.



That's it!

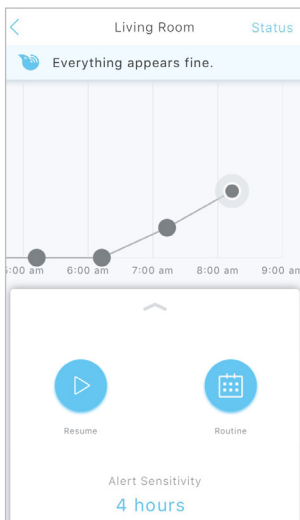

The AbiSensor's notifications are paused. While your AbiSensor will continue updating the graph every hour, all notifications of unusual activity from this AbiSensor will stop.

Working in the app, you have:

- Accessed the AbiSensor's graph page and paused notifications.

IMPORTANT

Remember to press **Resume** – to 'un-pause' the AbiSensor's notifications – when you are ready to receive notifications from the AbiSensor again (when your cared-for person is back home).

Related Guides

[How to Set or Change AbiBird Notification Preferences](#)